



Nitrile Glove Extend

Blue

White

Great stretch and good fit for precise comfort



Our product complies with HACCP, ISO 9001, ISO 13485, and CE marking requirements. We offer our customers a global stable supply of high quality gloves for medical and non-medical use.



View the introduction video of our facility

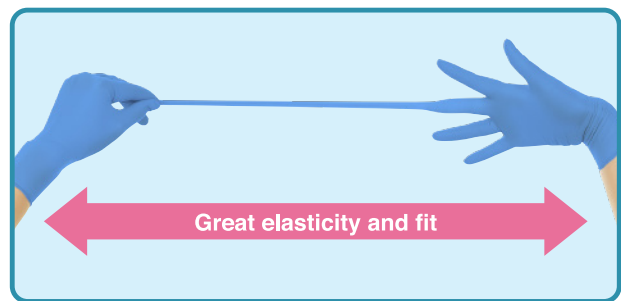
- HACCP
- ISO 9001
- ISO 13485

POWDER FREE

LATEX FREE

Features

- Great strength and perfect fit on hands
- Anti-slip emboss on fingertips
- Dispense glove from cuff. Enable hygienic dispensing solution
- High resistance to oil, chemicals, and heavy metals
- Complies with food safety laws



HACCP



ISO 9001



ISO 13485



Specification

Size	XS	S	M	L	XL
Length(mm)	≥ 240				
Palm width ±5 (mm)	75	85	95	105	115
Thickness(mm)	Fingertip : 0.10 ± 0.02				
	Palm : 0.08 ± 0.02				
Pinhole	AQL 1.5				
Tensile(MPa)	ASTM : Before aging ≥ 26 After aging ≥ 24				
Elongation(%)	ASTM : Before aging ≥ 500 After aging ≥ 400				
Force at break(N)	EN : Before aging ≥ 6 After aging ≥ 6				

Note: Comply with EN 455 and ASTM D6319

Product Performance

*Common to all sizes

Barrier performance

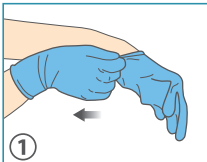
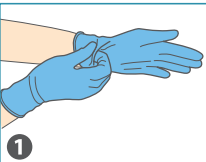
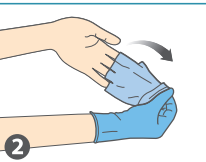
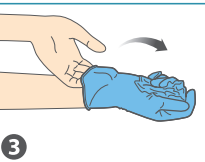
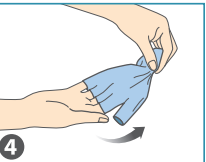

Tested chemical	Standard	Result
40% sodium hydroxide	: EN16523-1	>480 min
37% Formaldehyde	: EN16523-1	>120 min
30% Hydrogen peroxide	: EN16523-1	>30 min
Bloodborne Pathogen Penetration	: ASTM F1671 : EN374-5	Pass Pass

Skin irritation and sensitization : ISO 10993

Tested chemical	Result
Primary skin irritant Evaluation of irritation to skin contacted by the glove	Pass
Sensitization test Evaluation of skin sensitization (Type IV: delayed hypersensitivity immune reaction) by extracts from glove	Pass

Usage

*It is important not to touch the glove surface when removing them. Make sure to conduct correct hand hygiene after disposal.

Donning	Doffing				
					
Hold the cuff to put on	Pinch above the cuff on the surface	Turn the glove inside out while sliding it off	Insert the bare hand inside the cuff of the opposite glove	Turn the glove inside out while sliding it off	Dispose of gloves immediately after removing

SARAYA Nitrile Glove Extend

Blue



White



SARAYA

Saraya Co., Ltd.
2-2-8 Yuzato, Higashiumiyoshi-ku, Osaka, 546-0013, Japan
Phone: +81-6-6703-6336 URL: <https://saraya.world/>